



EMPTY TANK. DO NOT REFILL  
UNTIL BELOW 1000 PSI



FULL TANK AT 2000 PSI

### For tanks with BUILT-IN Regulators:



Turn OFF the tank before filling  
or when not in use



CONTINUOUS FLOW in WHITE  
ZONE  
2 LPM Oxygen flows  
continuously



PULSE FLOW 1 - 5 in BLUE  
ZONE  
Oxygen flows ONLY when you  
breathe in

Turn OFF the tank before filling, or when  
not in use, by rotating the valve handle  
CLOCKWISE until it stops.



For further instructions call 334-566-1002  
**NO SMOKING WITHIN 25 FEET OF OXYGEN EQUIPMENT**

## HOMEFILL INSTRUCTIONS

1. Turn **ON** the **Compressor** and let it warm up for **10 minutes**. *This is the machine on the bottom that makes the oxygen.* The flow meter must be below \_\_\_\_\_ to fill your tank.
2. Make sure that the **Tank** is turned **OFF** and the pressure is less than 1000 psi (see back of this page). Attach tank to the Compressor by inserting the brass connector **INTO** the brass connector on the **Compressor**. You will hear a “click” when the tank is properly connected.
3. Turn **ON** the **Compressor**. You will hear a “churning” noise as the tank fills. The yellow light will also stay on for roughly 5-10 minutes. *This is all normal!* Tanks, depending on size will fill in about an hour.



4. The **FULL** light will turn green when filling is complete. Turn the **Compressor OFF**.
5. Remove the tank by pressing **DOWN** on the white or silver connector while pulling the tank **UPWARD**.



# Jones Medical Supply

Home Oxygen & Medical Equipment



For further instructions call 334-566-1002  
**NO SMOKING WITHIN 25 FEET OF OXYGEN EQUIPMENT**